



SimplifEyes Daily Soft Contact Lenses **(42% Etafilcon A, 58% H₂O)**

These usage instructions apply to the following contact lens products: SimplifEyes™ daily soft contact lenses (42% Etafilcon A, 58% H₂O) immersed in phosphate buffered saline with Tangible® Polymers.

IMPORTANT INFORMATION WEARING ADVICE AND CARE INSTRUCTIONS FOR SOFT CONTACT LENSES

These instructions for use serve as a supplement to the recommendations of your eye care professional. They contain additional information for the safe usage of your contact lenses. Please read the instructions for use carefully.

Indications for Use: Indicated for the correction of ametropia (myopia, hyperopia, astigmatism, presbyopia) in aphakic and non-aphakic persons with non-diseased eyes.

Intended purpose: Your contact lenses are intended for vision correction (myopia, hyperopia, toric, presbyopia) of impaired vision determined by your eye care professional!

Your contact lenses must be adapted by an eye care professional when they are used for the first time or when they are changed (e.g., due to a change in visual acuity or a new type of contact lenses). Only use the care system recommended by your eye care professional and follow the instructions for use of the care system.

INSTRUCTION: Please remove and dispose of the lenses every day and use a fresh replacement contact lenses each day

Contact lens parameter specifications on the packaging:

Definition	Abbreviation
Base curve	BC
Contact lens diameter	DIA
Thickness of the cylindrical effect in diopters for compensation of corneal curvature/astigmatism	CYL
Near addition in diopters	ADD
Axis position of the cylindrical effect	AXIS
Contact lens thickness in diopters	PWR

Explanation of the symbols:

Definition	Symbol
Use by (month/year)	
Batch code	
Sterilized using steam or dry heat	
Do not reuse	
Caution	
Consult instruction for use	 SynergEyes.com
„L“ for left eye or „R“ for right eye as marked on the box label	
Should the safety closure be damaged or should there be no liquid in the container, please contact your eye care professional. Do not use if package is damaged	
Keep away from sunlight	
Manufacturer	
UV blocking	
Keep dry	
Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed eyecare practitioner.	
Trash box, litter bin, rubbish bin	
Blue tinted lens	

General information for all soft lenses:

The contact lenses are designed to be worn during the daily waking phase and may only be worn by the person for whom the lenses were adapted by an eye care professional.

Regarding the replacement interval, please observe the information specified on the packaging and the advice of your eye care professional. Never use the contact lenses for a longer period than specified, in order to avoid eye infections and tolerability problems. Store at room temperature!

Attention: Each time before touching contact lenses and eyes with your hands: Wash your hands thoroughly with a possibly perfume and oil-free soap and dry them with a clean and lint-free cloth. This allows you to minimize the transfer of germs and dirt onto the contact lens or your eye and avoid infections. Only touch the contact lens with your fingers.

Avoid sharp objects or long fingernails. If the contact lens is dirty, clean and disinfect the contact lens according to the instructions for use of your care system before putting the contact lens into your eye.

This does not apply to daily lenses since they are intended for single-use only.

Putting the contact lens into your eye: Remove the contact lens from the container. Place the contact lens on the fingertip of your index finger and check the lens for correct shape and damage (do not put the contact lens into your eye if it is damaged!). Look into a mirror and pull the lower eyelid down using your middle finger of the same hand. Pull the upper eyelid up using the middle finger of your other hand and carefully place the contact lens onto your eye. Release the eyelids slowly.

Visual quality: Your vision should be normal right after putting the lenses in place. Should you still have impaired vision, check the contact lens for correct positioning, possible dirt and side correctness or (in the case of different diopter values) check if the contact lens is placed into the correct eye.

Removing the contact lens from your eye: Slightly pull the lower eyelid downwards while looking upwards. Move the contact lens to the white part of your eye using your fingertip. If the contact lens cannot be moved, wet your eye with a suitable wetting solution. Grab the contact lens with your thumb and index finger. Make sure that your fingernails do not touch the contact lens or your eye. Do not use any other objects in order to remove the contact lenses.

PRECAUTIONARY MEASURES

- Consult your eye care professional before you start wearing contact lenses again after a longer period of disuse.
- Strictly adhere to the replacement interval of your contact lenses. Regarding the replacement interval, please observe the information specified on the packaging and the advice of your eye care professional.
- Some medications may impair the wearing of contact lenses. Inform your eye care professional on any changed or new use of medications.
- Do not expose the contact lenses to hairspray, cosmetic products or aerosol sprays. This also applies to working in environments contaminated with aerosols or chemical vapors. If a chemical substance comes into contact with your eye, remove the contact lens if required/possible, rinse your eye and immediately seek a doctor.
- Contact lenses may not come into contact with bath or swimming pool water, in order to avoid losing them, or contamination of the contact lenses which could result in a severe microbiological infection. Swimming goggles may prevent the contact lenses from coming into contact with water.
- Do not wear the contact lenses while sleeping unless recommended by your eye care professional.
- Inform your physician that you wear contact lenses.
- Inform your employer that you wear contact lenses.
- Do not exceed the wearing time recommended by your eye care professional. Comply with the recommended follow-up checks in order to find

possible ocular changes, and to adapt the handling recommendations accordingly.

CONTRAINDICATIONS

1. Do not wear contact lenses in the case of known, abnormal, recurring reactions of the anterior eye segment to wearing contact lenses.
2. In the case of dry eyes (insufficient tear film).
3. Do not use the product if you suffer from an allergy regarding contact lenses or ingredients specified on the contact lenses care product packaging.
4. Following eye surgeries or diseases, contact lenses may only be used again upon approval of a physician.
5. Your eye care professional should have informed you of possible contraindications.

WARNINGS

- Should the safety closure of the contact lens packaging or the contact lenses be damaged, the contact lenses must not be used. Dehydrated (dried-out) contact lenses must not be used; dispose of them immediately.
- For eye health purposes, it is important that the contact lenses can be moved easily on the eye. If your contact lenses cannot be moved on the eye, please contact your eye care professional.
- The UV protection of a contact lens does not replace the function of sunglasses.
- Remove the contact lens in the case of discomfort, eye redness, increased tear formation, foreign object feeling, itching, pain, blurred vision, light sensitivity or halo vision. If the discomfort cannot be removed by cleaning the contact lenses, remove them and immediately consult your eye care professional or eye specialist.

Cleaning and disinfection of weekly, monthly, semi-annual and annual contact lenses:

(For product identification purposes, please observe the specifications on the back of the contact lens packaging.)

In order to avoid eye infections, proper cleaning and disinfection of your contact lenses is required. Clean and disinfect the contact lenses once removed, using the multipurpose or disinfection cleaning solution for soft contact lenses. Only use the care system recommended by your eye care professional and follow the instructions for use of the care system. The container may become contaminated easily. Clean it at regular intervals and regularly replace it with a new one. Do not use tap water for the care of contact lenses and contact lenses containers.

Do not, under any circumstances, use tap water, saliva or other liquids than your contact lens care product for rinsing contact lenses or containers. This increases the risk of a severe microbiological infection.

Only use the disinfection and care system recommended by your eye care professional and follow the instructions for use of the care system.

If you do not wear your contact lenses daily, they must be cleaned and disinfected according to the used disinfection and care system before being worn again, in order to avoid infections. Otherwise, the contact lenses must be disposed of.

Also pay attention to the instructions for use of the disinfection and care system.

Wearing advice for single-use contact lenses (daily lenses):

(For product identification purposes, please observe the specifications on the back of the contact lens packaging.)

Daily lenses are intended for single use only. Daily lenses are not intended to be cleaned or disinfected for material and production-related reasons. An improper, repeated usage of daily contact lenses may result in severe, negative consequences for your eye health.

Distributed by: SynergEyes, Inc. Carlsbad, CA, USA
Made in R.O.C.